



Instructions

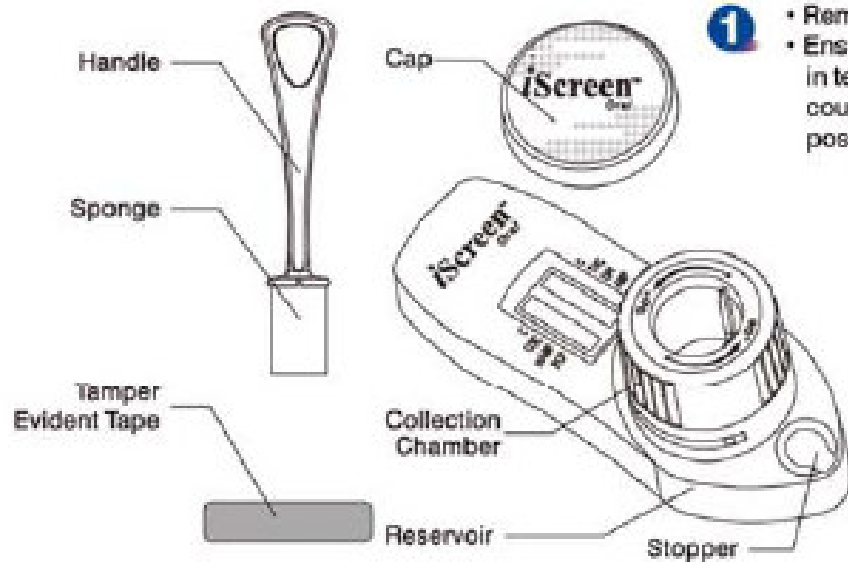


- 1**
- Remove cap.
 - Ensure reservoir is locked in testing position by turning counterclockwise as far as possible.


- 2**
- Insert sponge end of collector into mouth.
 - **Actively swab** the inside of the mouth and tongue with sponge. As soon as the sponge softens slightly, gently press the sponge between the tongue and teeth to ensure complete saturation.
 - The sponge is saturated when no hard spots can be detected. Collect for a total of (3) minutes before removing sponge.

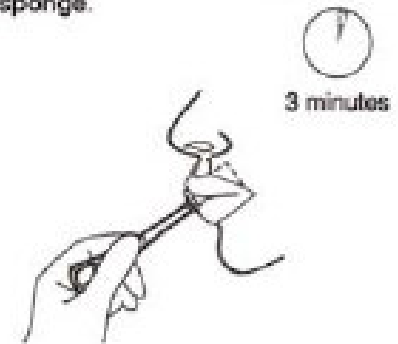

- 3**
- Insert collector into device as shown.
 - Push collector down into chamber.
 - Turn collector clockwise to lock into chamber.

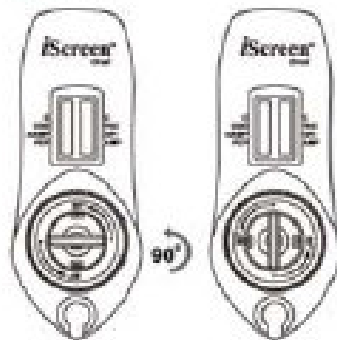

- 4**
- Wait 9 minutes, remove collector from device and read results.

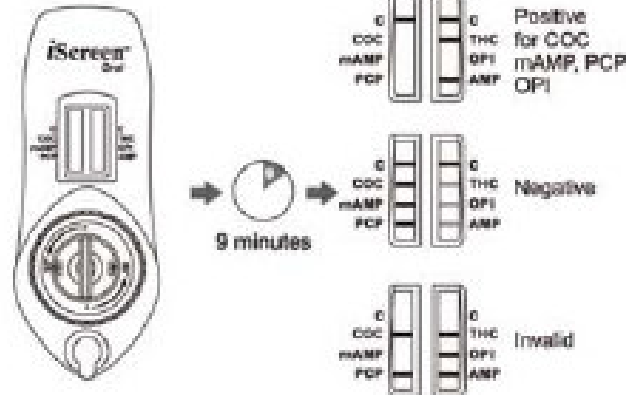

- 5**
- Secure cap back onto collection chamber.
 - Turn reservoir clockwise as far as possible to lock.
 - If results are positive, secure reservoir with tamper evident tape and send to laboratory for confirmation.

